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[www.techwinmed.com](http://www.techwinmed.com) info@techwinmed.com

Dear Specialist,

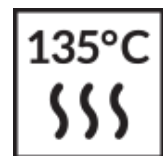
Thank you for choosing our product. We hope this tool contributes to improving the quality of treatment and your satisfaction. To ensure safe and efficient use, please carefully read the following guidelines. If you encounter any issues, do not hesitate to contact Techwin Dana Gostar customer service center.

## Technical Specifications

- Protection Type Against Electrical Shock: Class II
- Protection Against Electrical Shock: Type B
- Operation Mode: Continuous
- Liquid Protection: Normal liquids
- Operating Temperature Range: -40°C to +135°C
- Relative Humidity Range: 10% to 80%, (no condensation)
- Maximum Allowable Speed: 40,000 rpm
- Compatible Burs: Ø2.35 mm handpiece burs
- Head Compatibility: Accepts Doriot-type head

## Autoclave

Our product is designed to withstand the sterilization process in an autoclave at temperatures of up to 135°C. An autoclave is a highly effective method for eliminating bacteria, viruses, and other microorganisms using high-pressure steam, ensuring the highest standards of hygiene and safety .

## Important Warning

- Please read this manual carefully and keep it for future reference.
- This product is specifically designed for dental treatments and maxillofacial surgery; avoid using it for other purposes.
- Before operating the straight handpiece, ensure it is securely and firmly connected in place.
- Never operate the handpiece without a bur installed, as it may damage the head or motor.
- Avoid immersing the handpiece in liquids or chemical solvents, and do not use dry-heat sterilization.
- Sterilization using an autoclave at 134°C and 2 atmospheres of pressure is recommended.

### **Mounting the Bur on the Straight Handpiece**

To insert or remove a bur, move the lock-ring to the right until you hear a 'click,' then open the chuck. Insert or remove the bur as required. Move the lock-ring to the left until another 'click' confirms the chuck is securely locked. Ensure the bur rotates smoothly and is snug when twisted by hand before operating the device.

### **Tips for safe use and maximum service life**

- Always avoid using bent, off-centered, or damaged burs, as these may cause excessive loads, leading to premature wear of the ball bearings, especially at high speeds of 40,000 rpm.
- Always insert the bur fully into the chuck to prevent BUR-WALK-OUT and/or damage to the ball bearings.
- Never operate the straight nosecone without a securely inserted bur or bur blank.
- Ensure the Bur Lock Ring is correctly positioned before use.

### **Connecting the Straight Handpiece and motor**

Insert the handpiece onto the motor until you hear a soft 'click' sound. To remove it, gently pull the handpiece; it will come apart.

### **Cleaning the handpiece**

- Remove any dirt and residue from the handpiece.


- Stop the handpiece, clean it thoroughly, and dry it completely.

*Note: Use only clean water to clean the handpiece.*

### **Lubrication**

To maintain optimal performance, lubricate the handpiece after each use or before autoclaving:

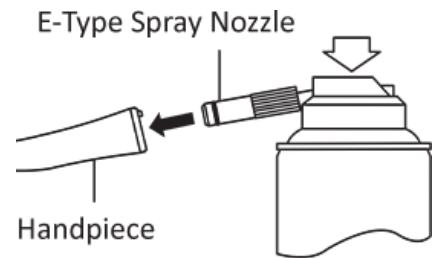1. Attach the E-type spray nozzle firmly to the spray nozzle.
2. Shake the can 3–4 times to mix the lubricant and propellant.
3. Insert the E-type spray nozzle into the rear opening of the handpiece and spray for 2–3 seconds until oil emerges from the head or nose of the device.

### **Cleaning and Sterilization**

#### **Cleaning**

- Clean the external surface of the handpiece with a soft cloth or cotton swab soaked in alcohol. Never use wire brushes for this purpose.
- Strictly avoid cleaning the handpiece in boiling water, chemical solutions, or ultrasonic cleaning devices.

#### **Sterilization**

Steam autoclave sterilization is recommended after every patient. Follow these steps:

1. Clean the handpiece sheath as outlined in the cleaning instructions.
2. Lubricate the device using spray (see 'Lubrication' section).
3. Place the handpiece into an autoclaving pouch and seal it per the pouch manufacturer's instructions.
4. Autoclave for 20 minutes at 121°C (250°F) or 15 minutes at 132°C (270°F) or 3 minutes at 134°C (275°F).

#### **Transportation and Storage Conditions**

- Environmental Temperature: -40°C to +50°C
- Relative Humidity: Less than 80%, no condensation

- Air Pressure Range: 0.5 atm to 1 atm

### **Guarantee Terms and After-Sales Service**

Techwin Dana Gostar guarantees your product under the following conditions:

1. This product is guaranteed for the repair or replacement of defective parts. The guarantee for bearings is 6 months from the date of purchase.
2. The guarantee is limited to the repair or replacement of defective parts, as determined by the manufacturer or authorized technical assistance network.
3. Failures caused by misuse, neglect of maintenance instructions, or damage from impacts are not covered under the guarantee.
4. Any repair or modification performed by unauthorized individuals will void the guarantee.
5. Guarantee repairs or replacements do not extend the original guarantee period .
6. Natural wear and tear are not covered under guarantee.
7. The customer is responsible for transportation costs related to guarantee claims.
8. This guarantee is valid only in the country where the product was purchased.
9. This product is covered by Techwin Dana Gostar's after-sales service for a period of 5 years.

Techwin Dana Gostar team expresses its gratitude to you, dear specialist, for choosing this tool and wishes you continued success!

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