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www.techwinmed.com info@techwinmed.com

Dear Specialist,

Thank you for choosing our product. We hope this tool enhances the quality and efficiency of your work. To ensure safe, efficient, and trouble-free use, please pay close attention to the following guidelines. If you encounter any issues, do not hesitate to contact Techwin Dana Gostar customer service center.

Contents of the Package


- 20:1 Contra Angle - 1 piece
- Cap Wrench - 1 piece
- Operation Manual - 1 sheet
- E-Type Oil Nozzle - 1 piece

Technical Data

- 55 N.cm \geq Torque
- Maximum Rotation Speed: 2000 rpm

Autoclave

Our product is designed to withstand the sterilization process in an autoclave at temperatures of up to 135°C. An autoclave is a highly effective method for eliminating bacteria, viruses, and other microorganisms using high-pressure steam, ensuring the highest standards of hygiene and safety.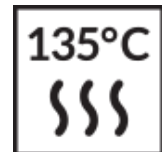

Application

This tool is designed specifically for professional dental surgeries in the field of implants.

Notes & Cautions

1. Before Operation

- This product is only compatible with ISO-standard implant systems. Using non-standard systems may damage the tool or pose risks to the patient.
- Only qualified professionals are authorized to use this tool.
- Before use, Check the functionality of the bur and chuck outside the patient's oral cavity. If there is looseness, excessive vibration, irregular noise, or overheating, do not use the handpiece.
- Avoid impacts, drops, and disassembly or modification of the handpiece.
- Keep at least one spare handpiece with the same functionality.
- Sterilize the handpiece before first use and after every treatment.
- Do not insert or remove the bur while it is in motion.
- Use only clean, standard drills. Any debris can cause vibration, reduced efficiency, or other issues.
- Do not exceed the maximum rotation speed of the bur or drill.

2. During Operation

- Do not operate the handpiece without a bur or drill installed.
- Never push the back cap while the handpiece is in motion, as this may cause damage or lead to the bur ejecting unexpectedly.


3. After Operation

- Clean, lubricate, and autoclave the handpiece after each use. Blood residue can corrode the cartridge.
- Lack of lubrication will cause the handpiece to heat up.
- Avoid using acidic solutions to scrub, inject, or soak the handpiece.
- Check functionality if the handpiece has not been used for an extended period.


Operation Instructions

1. Inserting or Removing a Bur/Drill

- **Inserting:** Press the push button firmly, insert the bur, and turn the rod clockwise until the bur is fully secured.

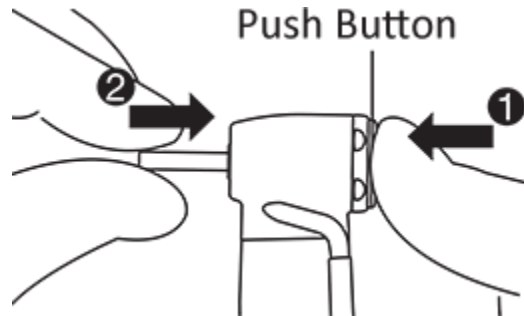
- **Removing:** Press the push button firmly and remove the bur.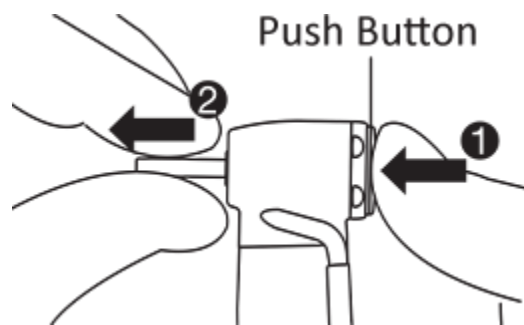

2. Attaching or Detaching the Handpiece

- **Attaching:** Insert the handpiece into the motor and turn it until it clicks into place.
- **Detaching:** Hold the handpiece and motor securely, then pull them apart.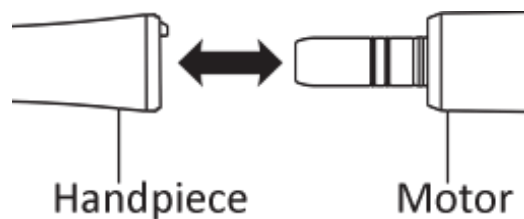

Note: Ensure the motor has completely stopped before attaching or detaching the handpiece.

3. Water Spray System


- **External Water Spray:** Connect the water pipe to the external water spray pipe on the handpiece.

Maintenance & Care



1. Cleaning

- Remove the bur from the handpiece after use. Brush it clean, soak the head in water, and operate it for 15 seconds before autoclaving.
- If the head is detachable, remove it and clean both the head and body with water.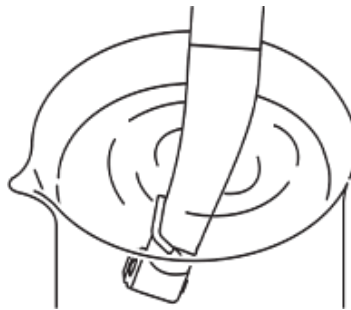

2. Lubrication

- Insert the E-type oil nozzle into the handpiece and spray for 2-3 seconds.
- If the head is dirty, remove the back cap with the spanner and spray oil inside.
- Lubricate the handpiece every 30 minutes during continuous use.

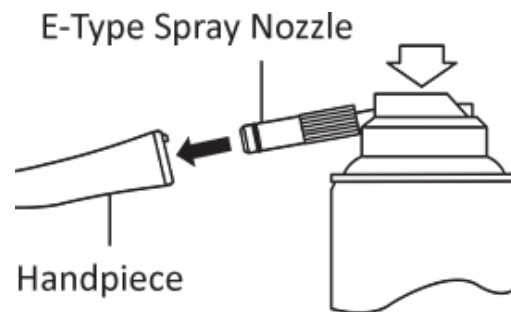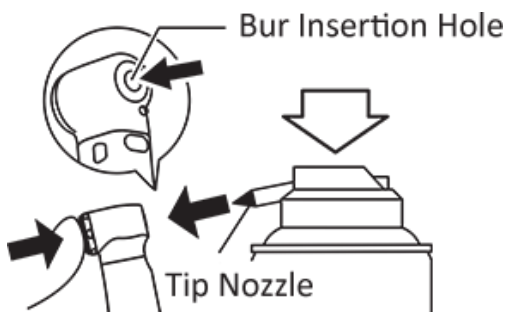3. Sterilization

Steam autoclave sterilization is recommended after every patient. Follow these steps:

1. Remove the bur and clean the handpiece, including the exterior, with an alcohol solution.
2. Lubricate the handpiece.
3. Place the handpiece into an autoclaving pouch and seal it per the pouch manufacturer's instructions.
4. Autoclave for 20 minutes at 121°C (250°F) or 15 minutes at 132°C (270°F) or 3 minutes at 134°C (275°F).

Note:

- *Do not use acidic solutions for cleaning, injecting, or soaking.*
- *Avoid autoclaving the handpiece with residual chemicals.*
- *Store the sterilized handpiece in a clean, dry, dust-free environment.*
- *Allow the handpiece to cool before touching it post-autoclaving.*

Guarantee Terms and After-Sales Service

Techwin Dana Gostar guarantees your product under the following conditions:

1. This product is guaranteed for the repair or replacement of defective parts. The guarantee for bearings is 12 months from the date of purchase.
2. The guarantee is limited to the repair or replacement of defective parts, as determined by the manufacturer or authorized technical assistance network.
3. Failures caused by misuse, neglect of maintenance instructions, or damage from impacts are not covered under the guarantee.
4. Any repair or modification performed by unauthorized individuals will void the guarantee.
5. Guarantee repairs or replacements do not extend the original guarantee period.
6. Natural wear and tear are not covered under guarantee .
7. The customer is responsible for transportation costs related to guarantee claims.
8. This guarantee is valid only in the country where the product was purchased.
9. This product is covered by Techwin Dana Gostar after-sales service for a period of 5 years.

Techwin Dana Gostar team expresses its gratitude to you, dear specialist, for choosing this tool and wishes you continued success!

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